Manufacturer

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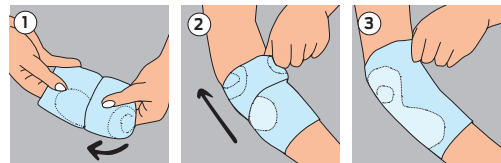
UK

Turn the bandage outwards so that the recesses of the profile inserts are visible.

Grasp the bandage in the area of the silicone pads and slip it over the forearm. The recess of the silicone insert must be in the direction of the outer bone protrusion.

Grasp the upper part of the bandage and pull it over the elbow. Make sure that the outer bone protrusion is positioned in the recess of the profiled insert. Pay attention to the positioning of the silicone pad!

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The Schapfix Epi Elbow brace is made from super elastic and breathable material which provides effective compression and **excellent comfort when worn**. Specially shaped visco-elastic pads relieve strain on pressure points in the elbow joint. Finally, its effective compression converts movement of the elbow into ongoing active massage of the surrounding tissues. Its great comfort allows this brace to be worn all day long!

Applying the Bandage *See page 7*

Care and Cleaning

Wash in warm water (by hand) with a mild detergent, rinse thoroughly and hang dry. Do not wash in the machine or dry clean. Do not tumble dry.

WARNING

- Do not wear on open wounds or if you have a tendency to skin disorder, allergic reaction or circulatory disorder.
- The bandage is intended for the individual care of the patient.
 - In case of improper use, product liability is excluded.
 - A combination with other products must be discussed in advance with the doctor.
 - If the symptoms increase or you notice abnormal changes, please consult your doctor.
 - Please do not allow the bandage to come into contact with fatty and acidic agents, ointments and lotions.

Not a substitute for medical care. Always consult a doctor for injury diagnosis and treatment.