Manufacturer

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The Schapfix OMO Shoulder brace is an active shoulder bandage for treatment of the shoulder joint. The anatomically structured knitted fabric enables a perfect body shape fit and guarantees **high degrees of comfort when worn**. If necessary a visco-elastic pad can be attached and positioned as required (this is included). The Schapfix OMO Shoulder brace supports, controls movement, promotes healing and helps to reduce pain, without restricting movement.

Applying the Bandage *See page 7*

Care and Cleaning

Wash in warm water (by hand) with a mild detergent, rinse thoroughly and hang dry. Do not wash in the machine or dry clean. Do not tumble dry.

- WARNING** Do not wear on open wounds or if you have a tendency to skin disorder, allergic reaction or circulatory disorder.
- The bandage is intended for the individual care of the patient.
 - In case of improper use, product liability is excluded.
 - A combination with other products must be discussed in advance with the doctor.
 - If the symptoms increase or you notice abnormal changes, please consult your doctor.
 - Please do not allow the bandage to come into contact with fatty and acidic agents, ointments and lotions.
- Not a substitute for medical care. Always consult a doctor for injury diagnosis and treatment.**